


# **FCR CAPSULA V VIEW**

## **Wersja 2.0**

# "CAPSULA V VIEW"

"Konsola" + "Przeglądarka" (opcja)

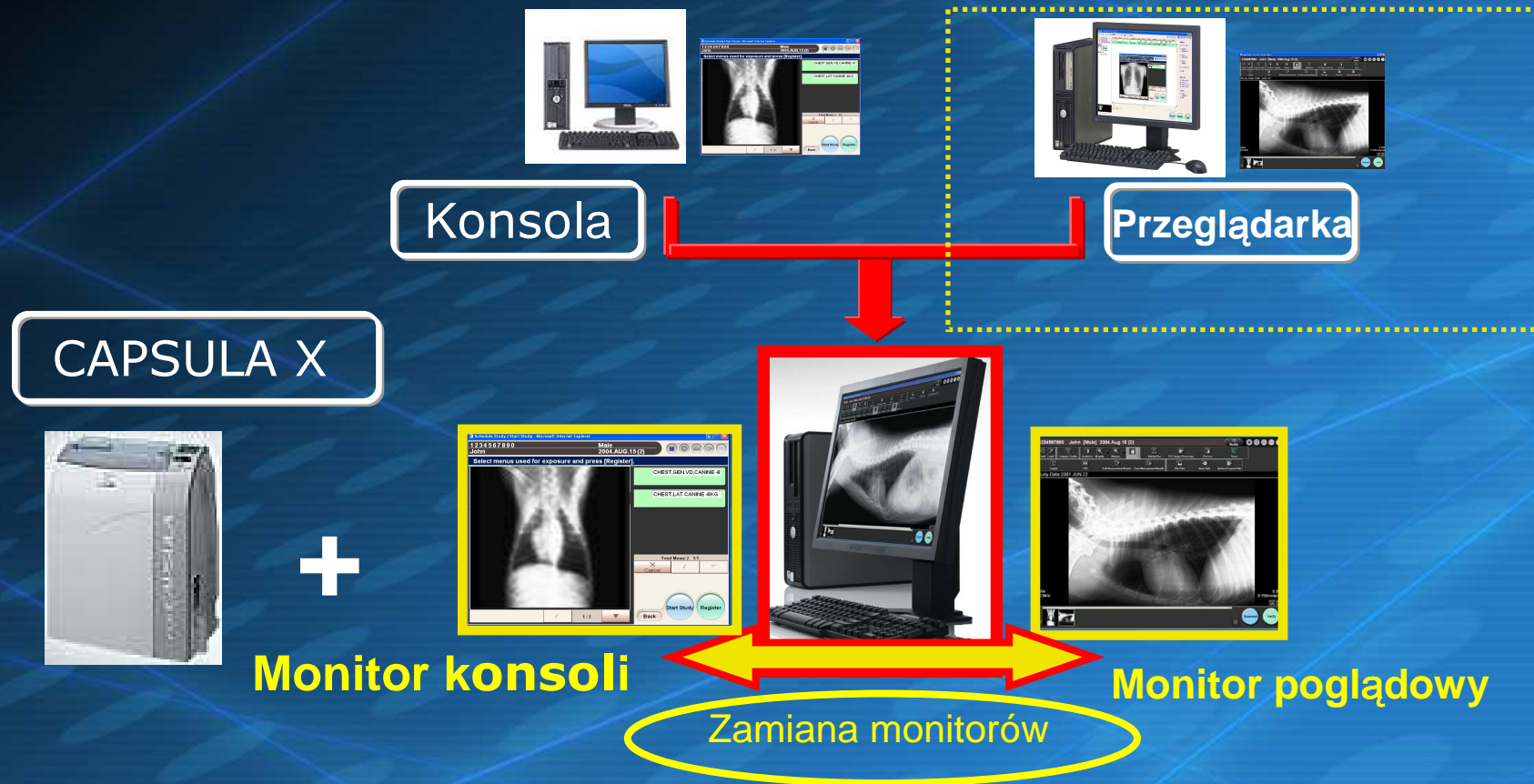

**CAPSULA V VIEW**

# "CAPSULA V VIEW"

CAPSULA V VIEW dzięki  
zaawansowanym  
funkcjonalnościom i  
wyjątkowej jakości  
obrazom zmieni stawianie  
diagnozy oraz zwiększy  
efektywność pracy.

FUJIFILM

FCR

See ahead with a digital view.

Image Intelligence

FUJI COMPUTED RADIOGRAPHY  
CAPSULA V VIEW **NEW**

FUJIFILM

# Praca z CAPSULA V VIEW ( 1 )

## ■ Wprowadzenie danych pacjenta

Schedule Study / Start Study - Microsoft Internet Explorer  
1234567890

Input patient information and press [Next].

Patient Name  
John

Sex  
Male Female Other

Date of Birth  
2004.AUG.15  
Entry  
Example>>1972.AUG.15

ABC Signs Space Delete char. Delete all. Back Next

## ■ Wybór projekcji

Schedule Study / Start Study - Microsoft Internet Explorer  
1234567890 Male  
John 2004.AUG.15 (2)

Select menus used for exposure and press [Register].

CANINE -4KG	CANINE -8KG	CANINE -12KG	CANINE -16KG
CANINE -20KG	CANINE +20K	FELINE -4KG	FELINE -8KG

CHEST,GEN.VD,CANINE -8  
CHEST,LAT CANINE -8KG

SKULL,NASALB.CANINE -8KG	SKULL,GEN.2R CANINE -8KG	CERVICAL,SOFT.2R CANINE -8KG
CHEST,GEN.(VD)2R CANINE -8KG	CHEST,GEN.(DV)2R CANINE -8KG	
ABDOMEN,GEN.(VD)2R CANINE -8KG	ABDOMEN,GEN.(DV)2R CANINE -8KG	
CERVICAL.SPN.2R CANINE -8KG	THORA.SPN.2R CANINE -8KG	LUMBAR.SPN.2R CANINE -8KG
SCAPULA.2R CANINE -8KG	UP.ARM.2R CANINE -8KG	ELBOW.JNT.2R CANINE -8KG
FO.ARM.2R CANINE -8KG	HAND.2R CANINE -8KG	
PELVIS,GEN.2R CANINE -8KG	THIGH.2R CANINE -8KG	THIGH,SOFT.2R CANINE -8KG
KNEE.JNT.2R CANINE -8KG	LO.LEG.2R CANINE -8KG	FOOT,2R CANINE -8KG


Total Menu: 2 1/1  
Cancel /

Back Start Study Register



# Praca z CAPSULA V VIEW ( 2 )

## ■ Ekspozycja

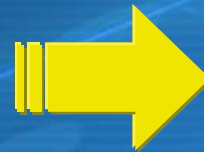


## ■ Odczyt płyty IP




# Praca z CAPSULA V VIEW ( 3 )

## ■ Kontrola obrazu, podgląd

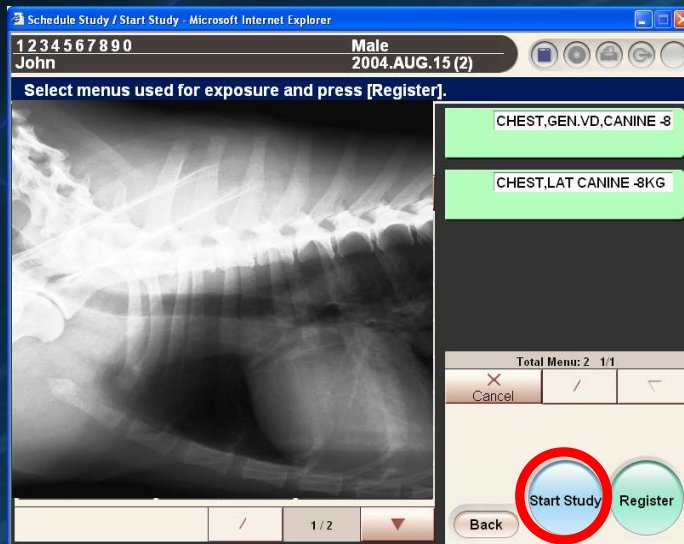

「Start Study」

## ■ Podgląd

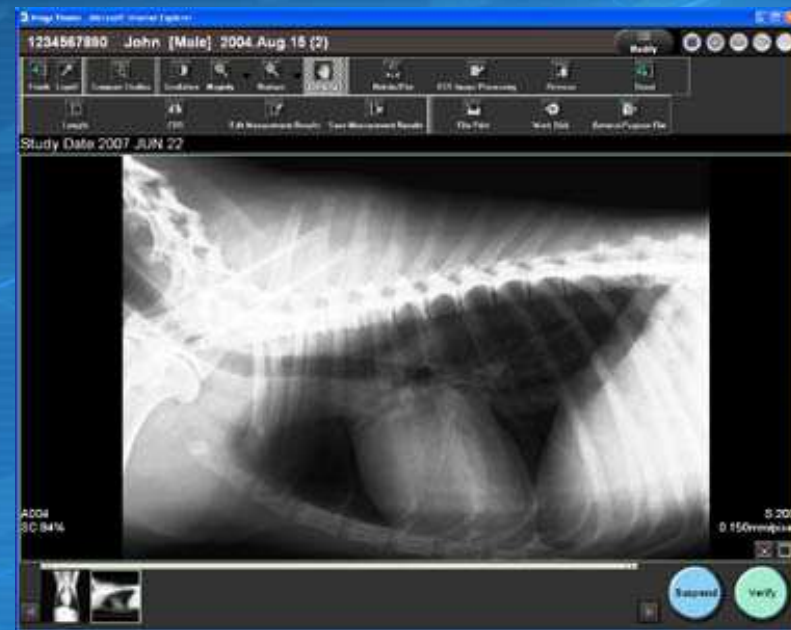

# Praca z CAPSULA V VIEW ( 4 )


## ■ Diagnoza



powiększ



porównaj obrazy

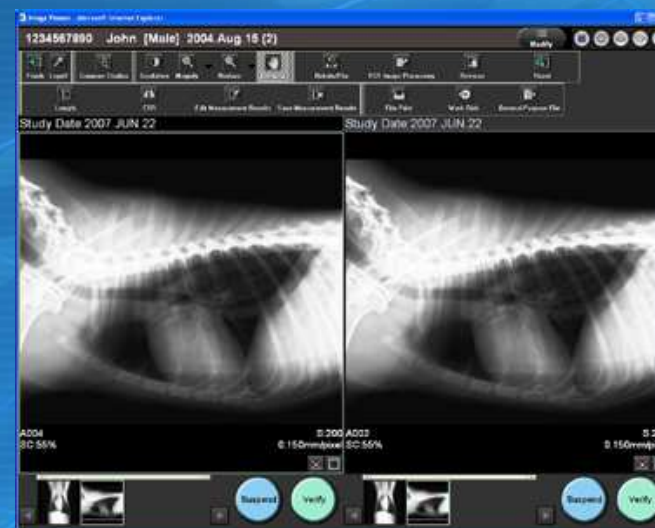

# Praca z CAPSULA V VIEW ( 5 )


## ■ Funkcje przeglądarki




porównaj obrazy



powiększ

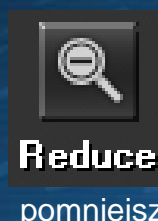

pomniejsz

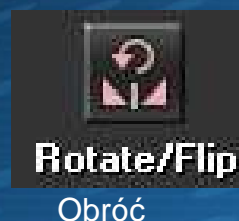

Obróć




drukuj



rozjaśnij



narzędzia postprocessingu

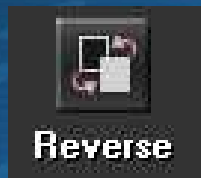

odwróć



cofnij




Pobierz z dysku



długość



CTR

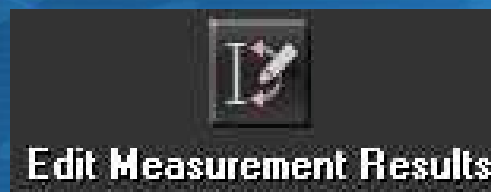

Edytuj pomiary



Zachowaj pomiary



Pliki


**FUJIFILM**

# Workflow of CAPSULA V VIEW ( 6 )

## ■ Informed consent



## ■ Record and Archive

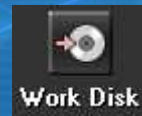

DVD+RW/-RW



**THE END**

**FUJIFILM**